

Exposé®

Disposable Caries Indicator

How about just staining caries?



Single-use applicators eliminate set-ups, cleanup and cross-contamination



With Exposé®, there is no measuring or mixing, and no need to stock vinegar for those messy accidents.

Exposé is a caries indicator built into an applicator and dispensing system! The detector solution is pre-applied to our Benda® Micro applicators. Apply Exposé simply by touching a wet tooth surface, and Exposé stains infected dentin. No dispensing, no dripping, no mess! Exposé eliminates the dreaded mess of bulk containers or brush-tipped syringes.

Features:

- No set-up, no cleanup, and no cross-contamination
- Supports minimally invasive dentistry
- Comes in two head sizes
- Available in 2 indicator colors: Blue and Red



Exposé is packaged in our Benda® Matic pop-up dispenser.



centrix®

Making Dentistry Easier.™


Instructions for use:



1. Remove an Exposé applicator from the Benda®Matic.



2. Lightly wet the Exposé applicator with either water or the wet tooth.

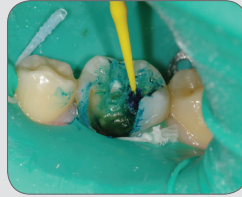

3. Apply the wet Exposé applicator to the prepared tooth.



4. Rinse and examine the area for any Exposé that remains on the affected dentin.



5. Proceed as usual.

Red Indicators



Fine (1.5 mm point)



Extra-Fine (1.0 mm point)

Blue Indicators



Fine (1.5 mm point)



Extra-Fine (1.0 mm point)



ORDERING INFORMATION

Exposé® Caries Indicator Kit

- 310322** Blue Indicator Extra-Fine, 96
Blue Indicator Fine, 96
Each in their own Benda Matic dispenser
- 310325** Red Indicator Extra-Fine, 96
Red Indicator Fine, 96
Each in their own Benda Matic dispenser



Exposé® Caries Indicator Refills

- 310330** Blue Indicator Fine, Yellow Handle, 576
- 310331** Blue Indicator Extra-Fine, Ivory Handle, 576
- 310332** Red Indicator Extra-Fine, Green Handle, 576
- 310333** Red Indicator Fine, Red Handle, 576
- 310334** Blue Indicator Fine, Yellow Handle, 100
- 310335** Blue Indicator Extra-Fine, Ivory Handle, 100
- 310336** Red Indicator Extra-Fine, Green Handle, 100
- 310337** Red Indicator Fine, Red Handle, 100

Ordering is easy. Call 800.235.5862 or contact your preferred dealer.
Visit centrixdental.com for more information.