

NoMIX[®]


Temporary Cement for Crowns & Bridges

Single component.
Moisture activated.
Direct dispensing.



NoMIX[®] Temporary Cement is easier for you...and your patients.

NoMIX is the only pre-mixed, single component, temporary cement available. It eliminates the mixing, spatulation and messy cleanup of traditional temporary cements. Yet NoMIX provides superior retention. Pre-loaded in syringes enables fast, exact placement onto restorations. And our new NoMIX take-home applicators are perfect for patient use – at home or on-the-go; they help eliminate those late night emergency calls.

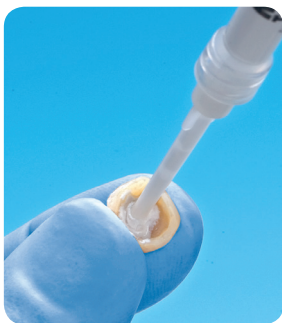

Features:

- NoMIX, no mess, no waste
- Easy to use, in the office or at home
- Non-eugenol, so it's compatible with all resin materials or bonding agents



centrix[®]

Making Dentistry Easier.SM

Benefits:

- **Moisture-activated formula** — no need to dry the tooth; reduces post-operative sensitivity
- **Prefilled 1mL syringe** — perfect for easy, direct application onto restorations
- **Unit-dose, take-home applicators** — take-home security; helps eliminate those late-night emergency calls
- **No eugenol** — won't inhibit resin materials from setting

Active Ingredients:

- **Potassium Sulfate** — desensitizing effect on the tooth
- **Zinc Oxide** — anti-bacterial ingredient, prevents the “black fungus” and corresponding bad breath
- **Calcium Sulfate** — expands slightly for minimal leakage

Working time 5 minutes. Final set time 15 minutes.

NoMIX Technique:

- Leave prepared tooth moist. Do not dry.
- Lightly cover the internal surfaces of the dry restoration with NoMIX Temporary Cement.
- Place restoration and check for fit. Have patient bite down.
- Wait 5 minutes. Cement will be set enough to gently remove any excess cement. Clean the interproximals by knotting a piece of floss and pulling through (NOT up).
- The cement will fully set in 15 minutes. Instruct the patient not to eat for at least 1 hour.

Place our new
Take-Home Display Kit
at your check out counter.
Every patient will leave
with extra security and
peace of mind!

ORDERING INFORMATION:

REF 310157	NoMIX Take-Home Display Kit 50 x 0.5g unit-dose take-home applicators in a convenient display box
REF 310520	NoMIX Starter Kit 6 x 1mL prefilled syringes 30 syringe tips 24 x 0.5g unit-dose take-home applicators
REF 290040	NoMIX Syringe Refill Kit 12 x 1mL prefilled syringes for use in-office
REF 640002	NoMIX Syringe Tips 30 syringe tips



Ordering is easy. Call 800.235.5862. Contact your preferred dealer.
Visit centrixdental.com for more information.

© 2016 Centrix, Inc. All rights reserved. NoMIX is a registered trademark of Centrix, Inc. MK0022 REV F