

Use Orotol® Plus daily and MD 555 weekly

After every treatment: aspirate 1 glass of water. Before every disinfection: aspirate 2 litres of water.

Daily use of Orotol® Plus (2%)

To be used after the end of treatment.
Where there are higher levels of contamination, use twice daily.



1. Aspirate 2 litres of water before disinfection.

2. Mix 2 litres of a 2% strength solution of Orotol® Plus

Fill the OroCup with cold water up to the 2 litre mark.

Add 2 dosing caps of Orotol® Plus (only fill up to the mark).



3. Shake contents in the OroCup.

4. Aspirate the solution using the OroCup and allow it to work overnight.

5. Disinfect the spittoon with the remainder of the solution (1 litre in the OroCup).

6. Next morning aspirate 2 litres of water.



Weekly use of MD 555 special cleaner (5%)

To be used once a week. Where there are higher levels of contamination, use daily before the lunch break.

1. Aspirate 2 litres of water before cleaning.

2. Mix 2 litres of a 5% strength solution of MD 555

Fill the OroCup with cold water up to the 2 litre mark.

Add 5 dosing caps of MD 555 or 1 sample bottle (only fill up to the mark).

3. Shake contents in the OroCup.

4. Aspirate the solution using the OroCup and allow it to work over the lunch break.

5. Clean the spittoon with the remaining 1 litre of solution.

6. Aspirate 2 litres of water after the lunch break.



Notice to the installer:

**Please ensure
this is given to the user
or left on display
with the pump.**