



Grandio® SO Inlay System

SET FOR CHAIRSIDE FABRICATION OF INDIRECT
COMPOSITE INLAYS

Grandio®SO Inlay System

SIMPLY COMPOSITE: THE INDIRECT IMMEDIATE INLAY FROM VOCO

You would like to offer your patients a restoration with affordable yet high-quality inlays which can be placed in just one session? You would like to avoid having to purchase expensive CAD/CAM equipment and still be able to generate the full added value within your practice? Then VOCO has the solution for you in the form of a unique complete set: GrandioSO Inlay System – the practical set for chairside fabrication of indirect composite inlays!

Quick and economical

In many respects, inlays offer huge advantages for the treatment of lesions in posterior teeth. Until now, however, fabricating inlays has almost invariably involved a time-consuming and costly procedure, requiring expensive equipment or the involvement of an external laboratory. Moreover, a temporary cover has to be placed on the prepared tooth and the patient must attend a second treatment session in the practice.

Thanks to GrandioSO Inlay System, it is now possible to fabricate first-class composite inlays chairside in a simple procedure in just one session. This enables you to retain the full added value in your practice, while at the same time making treatment pleasant and efficient for you and your patient: no temporary obturation, no second session and no second anaesthetic!

Inlays you can easily fabricate yourself

After preparing the cavity and taking an impression with an alginate of your choice, the GrandioSO Inlay System components can then be used. The procedure is as follows:

1. Construction of the model with the die silicone specially developed for this indication
2. Fabrication and finishing of the GrandioSO inlay
3. Bonding and luting with Futurabond DC and Bifix QM
4. Finishing and polishing with Dimanto, the one-step diamond polisher

An additional advantage of fabricating the inlay extraorally in the model is that you can create approximal contacts without having to use the time-consuming matrix technique. This enables you to carry out stress-free treatment with tooth-coloured and durable restorations even in patients who are reluctant to comply or non-compliant.

Perfectly coordinated components



Grandio®SO Inlay System

A HIGH-QUALITY COMPOSITE INLAY IN JUST A FEW STEPS

<p>Preparation / impression taking</p> <p>▶▶▶</p>			
<p>Die silicone</p> <p>▶▶▶</p>			
<p>GrandioSO</p> <p>▶▶▶</p>			
			
<p>Futrabond DC / Bifix QM / Dimanto</p> <p>▶▶▶</p>			
			

Grandio®SO Inlay System

SIMPLY COMPOSITE: THE INDIRECT IMMEDIATE INLAY FROM VOCO

Advantages

- Complete set for chairside fabrication of 15 composite inlays
 - full added value retained in the practice, no laboratory involvement
 - less expensive than CAD/CAM fabricated inlays
- Highest-quality, perfectly coordinated components
 - quick and simple fabrication of models
 - the premium composite GrandioSO guarantees durability and aesthetics
 - Futurabond DC and Bifix QM for secure bonding and luting
- Simple and efficient method
 - easy and convenient to fabricate, even for large multisurface restorations
 - fabrication of a row of inlays for treatment of neighbouring teeth
 - creation of approximal contacts without having to use the time-consuming and expensive matrix technique
 - perfect occlusal anatomy – modelled extraorally
 - stress-free fabrication of tooth-coloured and durable restorations, even in patients who are reluctant to comply or non-compliant



Presentations

- REF 2642 GrandioSO Inlay System – set
- REF 1186 Die silicone – cartridge 2 × 50 ml
- REF 2202 Mixing tips – 50 pcs. type 20

VOCO GmbH
 Anton-Flettner-Straße 1-3
 27472 Cuxhaven
 Germany
 Tel.: +49 (0) 4721-719-0
 Fax: +49 (0) 4721-719-140
 info@voco.com
 www.voco.com

Available from: